




# THE NEEDLE FREE ARTERIAL CONNECTOR (NIC)

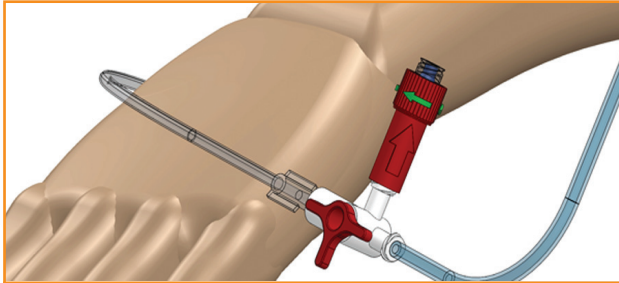


ATTACH CONNECTOR TO THE 3-WAY TAP

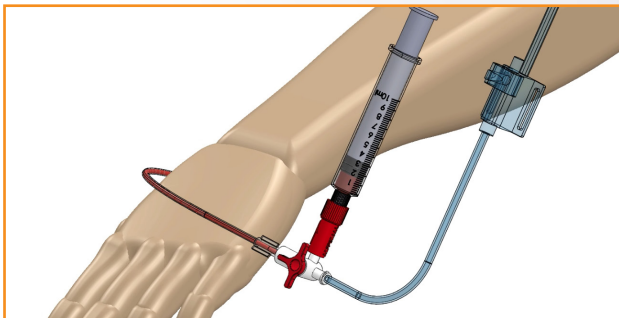

REMOVE DEAD SPACE

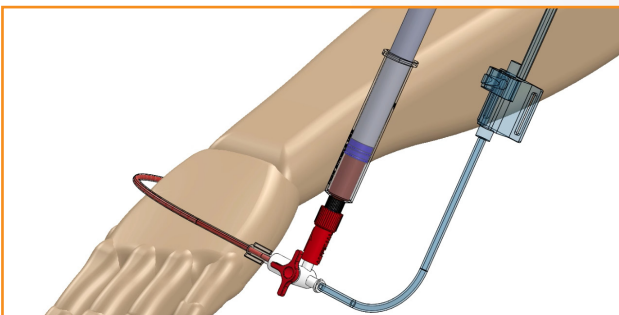

TAKE SAMPLE

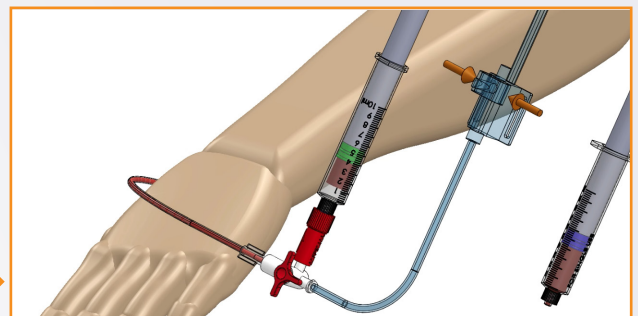

USING A FRESH SYRINGE FLUSH VIA  
TRANSDUCER UNTIL CLEAR FLUID IS OBSERVED.  
ENSURE THE SYRINGE DOES NOT FILL COMPLETELY

- Prevents accidental injection of medication
- Prevents blood spillage
- Prevents hub colonisation
- Prevents onward transmission of bacteria to patients

## Ordering Information:

Bunzl Product Code:  
**AMAMD031086**

Amdel Product Code:  
**AMD031086**

**AmDel**Medical  
A  **JAMES LYON & CO.** company