

Left Radial Support Sling Application

Pre-left radial access

1



Step 1. With patient supine put the plastic support under mattress at the level of the right elbow/upper arm. Loosely fix the narrow end of the sling to the plastic support using attached Velcro.

2



Step 2. Take wider end of sling across patient's body and place beneath the patient's left arm.

3



Step 3. Bring wider part of the sling into a loop around left arm and secure in position using attached Velcro.

4



Step 4. Patient can now be prepared using sterile drapes and radial access can be obtained.

Post-left radial access

5



Step 5. Operator/scrub nurse raises patient drape to allow a non-scrubbed member of staff to bring patient's left arm towards patient's abdomen.

6



Step 6. The sterile drapes are re-positioned to allow procedure to commence.

7



Step 7. The narrow end of the sling is adjusted as necessary to secure patient's left arm in required position. This position should be comfortable for patient and operator.

8



Step 8. Support board previously under patient's left arm during preparation can now be removed if required.